LANAFORM

COMFORT


Heating Blanket for Back

BACK & NECK HEAT PAD

EN	ENGLISH	BACK & NECK HEAT PAD
FR	FRANÇAIS	COUSSIN CHAUFFANT POUR LE DOS & LA NUQUE
NL	NEDERLANDS	VERWARMDE VEST VOOR DE RUG EN DE NEK
DE	DEUTSCH	WÄRMENDES RÜCKEN- UND NACKENPOLSTER
ES	ESPAÑOL	ALMOHADILLA PARA LA ESPALDA Y LA NUCA
IT	ITALIANO	CUSCINO TERMICO PER LA SCHIENA E IL COLLO
PL	POLSKI	URZĄDZENIE DO OGRZEWANIA PLECÓW I KARKU

CS	ČEŠTINA	VYHŘÍVANÁ VESTA PRO ZÁDA A ŠÍJÍ
SK	SLOVENČINA	VÝHREVNÝ VANKÚŠ NA CHRÁT A KRK
HU	MAGYAR	MELEGÍTŐ HÁT-ÉS NYAKPÁRNA
SL	SLOVENŠČINA	GRELNA BLAZINA ZA HRBET IN TILNIK
HR	HRVATSKI	GRIJAČI PRSLUK ZA LEĐA I VRAT
BG	БЪЛГАРСКИ	ЕЛЕКТРИЧЕСКА ГРЕЙКА ЗА ГЪРБА И ТИЛА
SV	SVENSKA	VÄRMEDYNA FÖR RYGG OCH NACKE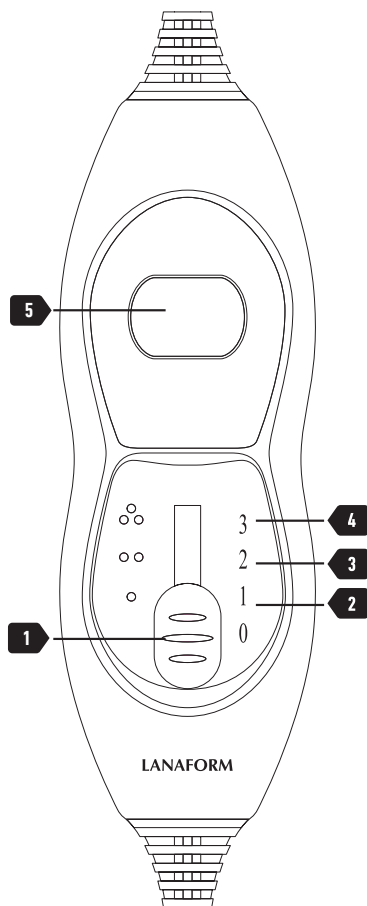

☒ The photographs and other images of the product in this manual and on the packaging are as accurate as possible, but do not claim to be a perfect reproduction of the product.

Thank you for choosing the Lanaform Heating Blanket for Back heated vest.

Lanaform's Heating Blanket for Back heated vest, has undergone the rigorous testing required under current European standards. Furthermore, the heated vest has phased temperature control that lets you warm yourself in winter and in damp weather.

⚠ Please read all the instructions before using your Heating Blanket for Back, especially this basic safety information

⚠ To prevent the risk of burns, electrocution, fire or injury:

Before connecting the device, check that your mains voltage is the same as that of your device.

Only use this device according to the instructions given in this manual. **This** appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.

Children shall not play with the appliance. **C**leaning and user maintenance shall not be made by children without supervision.

This heat pad must not be used by children under 3 years old because they might not react if it overheats. **T**his heat pad must not be used by young children (between 3 and 8 years old) unless a parent or a guardian has set the power switch and the child has been given enough information to use the heat pad safely. **D**o not use accessories that are not recommended by Lanaform or that are not supplied with this device.

If the power cable is damaged, it must be replaced by a special or similar cable available from the supplier or the latter's after sales department. **D**o not use this device if the power plug is damaged, if the device is not working properly, has been dropped on the floor or if it is damaged or has been dropped into water.

Have the device examined and repaired by the supplier or its after-sales department. **D**o not carry this device by its power

EN

3 - ENGLISH

cable or use the cable as a handle. **A**lways unplug the device after use or before cleaning. **U**nplug a device that has fallen into water immediately before retrieving it. **A**n electrical device must never remain plugged in without supervision. **U**nplug it when you are not using it. **U**se your heated vest only with the electrical plug provided. **D**o not fold the power cable and do not use the heated vest when it is creased, rolled up or folded. **N**ever use this device in a room where aerosols (sprays) are used or in a room or where oxygen is being administered. **T**o disconnect the device, turn the button to the "OFF" position, then remove the plug from the mains. **D**o not use safety pins, do not scratch the heated vest with a sharp object and do not beat it with a blunt object. **D**o not attach the device with pins or other metal objects. **N**ever let anything drop or insert any object into any openings. **D**o not expose your heated vest to the sun and do not iron it. **T**his will prevent damage to the insulation of the electrical circuit. **T**his device is not intended to be used

in a hospital environment, but for use in the home only. **D**o not use the heated vest with other heating devices or to heat other devices (hot-water bottle, electric blanket, etc.). **I**f you are concerned about your health, consult a doctor before using this appliance. **I**f you experience any pain when using this device, stop using it immediately and consult your doctor. **N**ever use this device on a body part that is swollen or inflamed or if you have a skin rash. **T**he remote control must remain within reach, do not leave the switch hidden under a pillow or any other object. **D**o not wash the heated vest too often to avoid excessive wear and tear. **T**he heated vest should not be washed in the washing machine more than 5 times during its life. **D**o not use the device when it is damp and do not turn it on to dry it. **W**hen storing your heated best for a long period, store it in a dry place. **C**arefully check its condition before operating it again for the first time. **I**f the power cable appears damaged, do not operate the device and take it back to the supplier or its

EN

after-sales department. This device should not be used by young children, people who are insensitive to heat or on animals. The device should not be used on a folding bed or on a bed, a section of which can be adjusted. Examine the device frequently to check for any signs of damage to the product or signs of wear. If you notice signs of wear or damage, stop using this product immediately and take it back to the supplier or the its after-sales department.

Using the electric heated vest

Diagram and description

- 01 OFF button
- 02 Position "1" (1st setting)
- 03 Position "2" (2nd setting)
- 04 Position "3" (3rd setting)
- 05 Display screen

Operating the heated vest

- Connect the plug to the socket at the bottom of the heated vest and the other end to an electricity supply (AC220-240V). Make sure that there is a power socket nearby and that the power cable is not in a position where the user may trip over it.
- Put on the electric heated vest using the tether strap and velcro for the neck. The light side of the vest should be against your back.

- When it lights up, the power status display lights up.
- Select the heat output required (1 to 3) using the heat control button.
- To turn the device off, turn the button to the "Off" position.

Note: The Heated Vest is set to run continuously for 90 minutes. If you wish to extend that time, it is advisable to let the heated vest cool down for 5 minutes before using it again.

Cleaning and maintenance

When you are not using the electric heated vest and before cleaning it, follow the instructions below:

- Make sure the control button is in the "OFF" position.
- Disconnect the power supply.
- Disconnect the plug linking the control to the device.
- Let the heated vest cool down before cleaning.

Cleaning

Small stains can be cleaned with a cloth or a damp sponge and, if necessary, with a little liquid detergent for delicate fabrics.

Warning: the electric heated vest should not be dry-cleaned, bent, dried in a dryer or machine-pressed or ironed.

If the heated vest is very dirty, it can be washed in a washing machine. Set it to the 30°C programme for delicate fabrics. Use a detergent for delicate fabrics and measure it out according to the manufacturer's instructions.

Do not wash the heated vest too often to avoid excessive wear and tear. The heated vest should not be washed in the washing machine more than 5 times during its life.

When drying the heated vest, do not hang it up with clothes pegs or other such devices.

Do not expose the heated vest to heat sources such as the sun, a radiator, etc.

Only reconnect the power cable to the heated vest when the plug and the vest are completely dry.

⚠ Never switch the electric heated vest on to dry it.

Storage

If you are not using the heated vest for a long time, it is advisable to store it in its original packaging in a dry place and not to place any objects on top of it.

Advice regarding the disposal of waste

FR All the packaging is composed of materials that pose no hazard for the environment and which can be disposed of at your local sorting centre to be used as secondary raw materials. The cardboard may be disposed of in a paper recycling bin. The packaging film must be taken to your local sorting and recycling centre.

When you have finished using the device, please dispose of it in an environmentally friendly way and in accordance with the law.

Limited warranty

Lanaform guarantees this product against any material or manufacturing defect for a period of two years from the date of purchase, except in the circumstances described below.

The Lanaform guarantee does not cover damage caused as a result of normal wear to this product. In addition, the guarantee covering this Lanaform product does not cover damage caused by abusive or inappropriate or incorrect use, accidents, the use of unauthorized accessories, changes made to the product or any other circumstance, of whatever sort, that is outside Lanaform's control.

Lanaform may not be held liable for any type of circumstantial, indirect or specific damage.

All implicit guarantees relating to the suitability of the product are limited to a period of two years from the initial date of purchase as long as a copy of the proof of purchase can be supplied.

On receipt, Lanaform will repair or replace your appliance at its discretion and will return it to you. The guarantee is only valid through Lanaform's Service Centre. Any attempt to maintain this product by a person other than Lanaform's Service Centre will render this guarantee void.

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